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Fimlab

CONTENTS OF THE SAMPLING KIT:

A sample transport tube with a sampling stick inside.

- HPV self-sample cannot be performed while you are on your period.
- Hysterectomy does not prevent self-sampling.
- If you are under any topical treatment (such as vaginal suppositories), be sure to stop using them on the sampling day and the day before, otherwise you can continue using them normally.
- Any hormone replacement therapy does not need to be interrupted.
- Normal pregnancy does not prevent self-sampling until week 35 of pregnancy. During the postpartum period, you can check the instructions for self-sampling with your clinic.

Collect and return the sample within 3 weeks of receiving the sampling kit.

1. Wash your hands. Open the sample transport tube just before use by twisting the red cap with enough force to break the seal. Do not remove the entire seal. Do not touch the soft tip of the sampling stick and be sure not to touch any surfaces with the tip. Hold the sampling stick in your hand below the red line.
2. Spread the labia around your vaginal opening using your other hand. Insert the sampling stick into the vagina to a depth of about 5 cm. Rotate the sampling stick inside the vagina for about 30 seconds, making sure that it is rubbing the vaginal walls. Gently pull out the sampling stick. **Be sure not to place the sampling stick anywhere or touch anything with it.**
3. **Insert the sampling stick back into the sample transport tube and close the cap by pressing on it.**
4. Write sampling date on the adhesive label on the preliminary information form, remove the label and attach it to the sample transport tube so that the bar code is positioned vertically.
5. Complete the preliminary information form either electronically at www.fimlab.fi/hpvomaseulontalomake or on paper (the form is included in the sampling kit).
6. Put the sample transport tube – and the preliminary information form if you completed it on paper – in the return envelope. Mail the return envelope within 24 hours. The postage has been paid. You will receive the results by mail (or to your OmaPosti service if you have it in use).

Further information: Fimlab Laboratoriot Oy customer advice service tel. 010 808 515, weekdays 7:00-16:00.

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Mer information: Fimlab Laboratoriot Oy:s kundrådgivning tfn 010 808 515, vardagar kl. 7-16.